

Clinicide

Broad Spectrum Germicidal Detergent and Deodorant

DIN 01918060



INDICATIONS

Clinicide is a concentrated, multi-purpose germicidal detergent and deodorant that disinfects, cleans and deodorizes in one step.

DESCRIPTION

- Broad spectrum of kill-bacteria, fungal and viral
- Pleasant smell, thus appealing to client in veterinary clinic
- Economical; 1:28 dilution

PACKAGING

ITEM NO.	UNIT PACKAGE	CASE SIZE
1CL1003	4 L	4
1CL1001	20 L	1

See reverse side for Administration and Dosage.

Clinicide

Broad Spectrum Germicidal Detergent and Deodorant

DIN 01918060

Virucidal • Bactericidal • Fungicidal **For Veterinary Hospital and Animal Facility Use**

Active Ingredients:

Didecyl dimethyl ammonium chloride 4.61% w/v
n-Alkyl dimethyl benzyl ammonium chloride
..... 3.07% w/v
Concentrate: Dilutes 1:128, one ounce per 128 ounces
of water or 8 mL Clinicide per 1 litre of water.

Clinicide - Recommended for use in hospitals, colleges, veterinary clinics, animal life science laboratories, pet shops, kennels, breeding and grooming establishments.

Clinicide is a concentrated, multi-purpose germicidal detergent and deodorant that disinfects, cleans and deodorizes in one labour saving step.

Clinicide has been tested in up to 400 p.p.m. hard water (calculated as CaCO₃) plus 5% organic serum and shown to be effective against the following organisms according to current test methods at a dilution of 1:128:

Bactericidal

Pseudomonas aeruginosa
Staphylococcus aureus
Salmonella choleraesuis
Enterobacter cloacae
Streptococcus pyogenes
Streptococcus faecalis
Enterobacter aerogenes
Salmonella typhimurium
Klebsiella pneumoniae
Proteus vulgaris
Serratia marcescens
Shigella flexneri
Shigella sonnei
Salmonella typhi
Proteus mirabilis
Escherichia coli

Fungicidal

Trichophyton interdigitale
Candida albicans

Virucidal

Influenza A / Hong Kong
Herpes Simplex types I & II

Vaccinia
Rubella
Adenovirus type 4
Feline picornavirus
Feline Leukemia
Canine distemper
Rabies
Pseudorabies
Avian IBV

Directions For Use: For use on hard non-porous surfaces such as floors, walls, metal surfaces, porcelain and plastic surfaces. Remove heavy soil deposits, then thoroughly wet surfaces. Use 8 mL Clinicide per litre of water for a minimum contact time of 10 minutes in a single application. Can be applied with a mop, sponge or cloth as well as spraying or soaking. Prepare a fresh solution for each use. Rinsing is not necessary unless floors are to be waxed or polished.

Precaution: Avoid contact with eyes. Harmful if swallowed. Do not contaminate water, food or feed by storage or disposal. Do not use empty container for other purposes.

First Aid Treatment: In case of contact with skin or eyes, immediately flush with plenty of water for at least 15 minutes. For eyes, call a physician.

Storage: Store at room temperature.

8CLI201-909
1CLI003

8CLI201-909_Clinicide_4L_Tech_03/11